
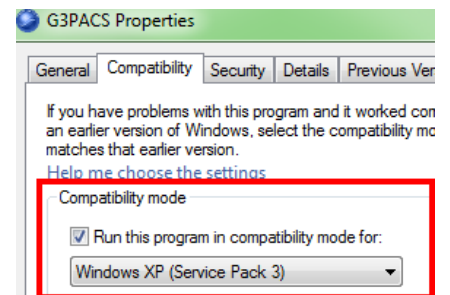


GO Imaging - PACS Install Guide

<https://pih.infinittna.com>

You will need Administrator privileges on your computer to install INFINITT PACS.

- Open **Microsoft Internet Explorer** and enter the web address shown above.
 - Click the *Tools* menu and then choose *Internet Options*.
- On the **General** tab click on *Browsing History Settings*.
 - Choose *Every time I visit the webpage* and click *OK*.
- Go to the **Security** tab.
 - Select the *Trusted Sites* zone and then click on the *Sites* button.
 - Add this website to the zone: <https://pih.infinittna.com> - then click *Add*.
 - Click the *Custom level* button for the *Trusted Sites* zone.
 - Reset custom settings to **Low**. Hit the **Reset** button and then *OK* to exit.
- Go to the **Privacy** tab.
 - Turn OFF the pop-up blocker.
- **Internet Explorer 10 and higher:**
 - Click the Compatibility Mode button (broken page icon). 
- **Windows 7 and Vista Pro users:**
 - Go to the *Control Panel > User Accounts*. Change *User Account Controls* settings to Never Notify.
- **Windows 8 users:**
 - Go to the folder *C:\Infinitt\Viewer* and find the *G3PACS.exe* program.
 - Right click on the icon and select *Properties*.
 - Choose the *Compatibility* tab and select the box to *Run this program in compatibility mode for:*



You are ready to use INFINITT RIS/PACS. Enter your *User ID* and *Password* on the login screen. Smart Update will run and install the necessary software. A shortcut will be installed on the desktop for quick login access.

System Requirements:

Operating System:	<i>Windows XP Pro , Windows Vista Pro, Windows 7 (any version)</i>
Monitor resolution:	<i>1024 x 768 (minimum) , 1280 x 1024 and higher (recommended)</i>
Web Browser:	<i>Internet Explorer versions - 7 , 8, 9, 10 (Internet Explorer ONLY)</i>
Computer Specs:	<i>1GB RAM (minimum) , 10GB hard drive space , High Speed Internet</i>


Accessing GO Imaging PACS

Note: If you do not see the Infinitt desktop icon, contact your PACS administrator.

- Double click the **INFINITT GO Imaging** icon on the desktop.
- Enter the User ID and Password that was given to you by your PACS administrator.
- The first time you login to PACS, you will be prompted to change your password.
- After you have entered the old and new passwords, click the Modify button.




The screenshot shows the INFINITT login interface. At the top left is the INFINITT logo. The main content area is a dark grey box with a key icon and the word "Login". Below the icon are two input fields: "UserID" and "Password". To the right of the "Password" field is a "Login" button. At the bottom of the box, there is a small copyright notice: "COPYRIGHT (C) INFINITT, CO., LTD. ALL RIGHTS RESERVED."

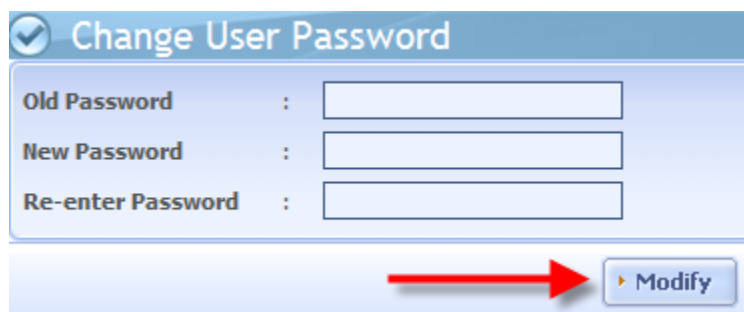

The screenshot shows the "Change User Password" interface. It has a title bar with a checkmark icon and the text "Change User Password". Below the title bar are three input fields: "Old Password", "New Password", and "Re-enter Password". At the bottom right, there is a "Modify" button with a red arrow pointing to it.

Infinitt PACS Worklist

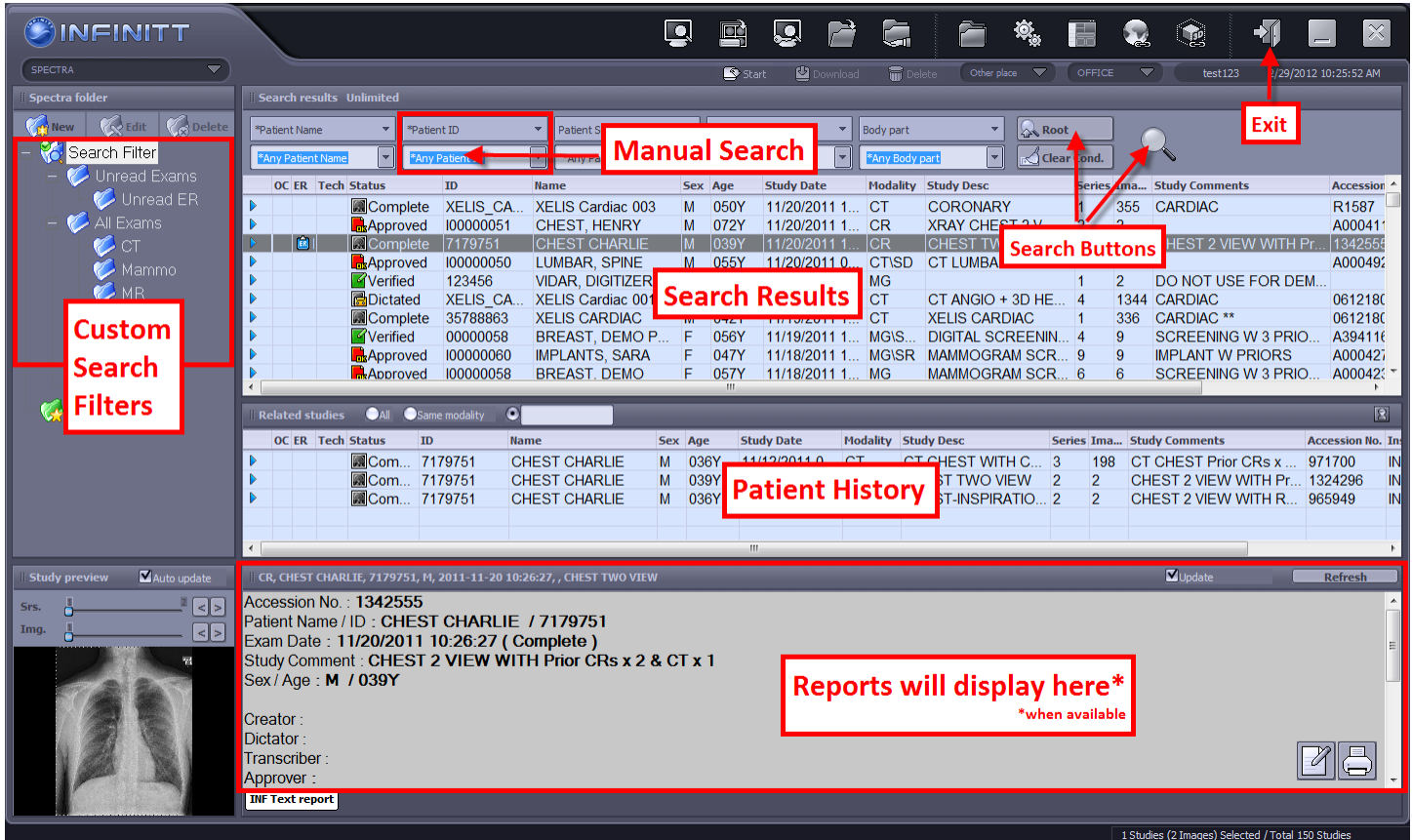

The screenshot shows the Infinitt PACS interface with several key areas highlighted:

- Custom Search Filters:** A sidebar on the left containing folders like 'Unread Exams', 'Unread ER', 'All Exams', 'CT', 'Mammo', and 'MR'.
- Manual Search:** Search input fields for Patient Name, Patient ID, and Body part.
- Search Results:** A table listing search results with columns for OC, ER, Tech, Status, ID, Name, Sex, Age, Study Date, Modality, Study Desc, Series, and Accession.
- Patient History:** A section below the search results showing a list of related studies for a selected patient.
- Reports will display here*:** A section at the bottom showing a study preview with patient information and a chest X-ray image.
- Search Buttons:** Buttons for 'Root', 'Clear cond.', and a magnifying glass icon.
- Exit:** An 'Exit' button in the top right corner.

Custom Search Filters

You can click on a folder and the *Search Results* will update according to that custom filter. To refresh the search results, click on the folder again.

Manual Search

Use this to search for a specific Patient Name, Patient ID, Accession #, Study Date, etc.

Search Results

Here you can see the patient and exam information for the current search.

Patient History

When you single click an exam in the *Search Results* window, the patient's history (images) will display here.

Report Display

If a report is attached to an exam, you can single click on it from the *Search Results* or *Patient History* window to view the radiology report.

Infinitt PACS Viewer



Using the Tool Set

Select a tool by left clicking on it. With the tool selected, move the mouse over the image and hold the left mouse down while moving the mouse in the desired direction.

Window Level / Width (By default your right mouse button will control windowing)

Holding down the right mouse button while you move the mouse up, down, left, right over the image will adjust the window level and width.

Crosslink

Use this feature to link axial, coronal and sagittal series and to show scout lines. This is useful for MRI and CT studies.

ALL

Switch Up/On = modify all images in a series (ex. window level all images).
Switch Down/Off = individually modify one image at a time in a series.

Exam(Original)

Click this drop down menu for a list of prior/related exams.